



Scuba Spec, Inc.
220 Richland Ave., W.
Aiken, SC 29801
1-800-457-5155
Fax: 803-643-8056

SCUBA SPEC
137EBP

The **SCUBA SPEC 137EBP**, or **Snorkel Spec**, is an underwater spectacle kit designed specifically for the visually deficient diver or snorkeler. The same prescription (Rx) power as used in the diver's habitually worn glasses (including bifocals, if desired) is used. There is no vertex variation. The Snorkel Spec can be easily inserted and removed from the mask at the diver's discretion.

The 137EBP is recommended for snorkeling or shallow water diving (<30 feet depth) only. It is not recommended for deeper dives.

INSTALLATION AND CARE INSTRUCTIONS

The Snorkel Spec is temporarily 'suctioned' or to the face plate using a small suction cup. To ensure proper suction, glass contact surface must be thoroughly cleaned. It is recommended that you use water or a mild soapy solution. Do not use window cleaner or an alcohol-based solution. Always moisten the inside of the suction cup prior to pressing it onto the face plate.

To prevent fogging of the face plate and lenses, a good anti-fog preparation should be applied before diving. Inquire about same from your eye care professional or dive shop. An alternative surfactant such as Lemon Joy detergent can also be used. Apply the detergent to the glass surfaces and briskly 'polish on' until the surfaces are 'squeaky clean'.

It is important that the nylon Snorkel Spec be hydrated every 2-3 months by soaking in water for 24 hours.

NOTE TO OPTICIAN:

- Lenses should be edged according to the "R" and "L" eye designations imprinted on the back of the frame.
- The nylon frame should be hydrated every 2-3 months by soaking in water for 24 hours. This will help maintain the frame's flexibility.

FRAME WARRANTY (excluding lenses): 4 years against breakage if properly cared for. Proper care entails hydrating (soaking) the nylon frame in water for 24 hours every 2-3 month.