



Scuba Spec, Inc.
220 Richland Ave., W.
Aiken, SC 29801
1-800-457-5155
Fax: 803-643-8056

**COMPLETE
SCUBA SPEC
UNIT**

The **Complete Scuba Spec Unit** is a dive mask with Scuba Spec (#199EB) installed. Specifically designed for the visually deficient diver, the corrected frame front can be snapped in and out of the mask at the diver's discretion. The same prescription (Rx) power as in the diver's habitually worn glasses (including bifocals, if desired) is used in the Scuba Spec. There is no vertex variation.

CARE AND USE INSTRUCTIONS

- When cleaning the surface area of the faceplate, it is recommended that you use water or a mild soapy solution. Do not use window cleaner or an alcohol-based solution.
- To unsnap the frame from the anchor bracket, grasp each eye rim between the thumb and middle finger and gently rock the frame off of the bracket.
- To prevent fogging of the face plate and the lenses, a good anti-fog solution should be applied before diving or snorkeling. Inquire about same from your eye care professional or dive shop. An alternative surfactant such as Lemon Joy detergent can also be used. Apply the detergent to the glass surfaces and briskly 'polish' it on until the glass surfaces are 'squeaky-clean'.

NOTE TO OPTICIAN:

- The "R" and "L" eye designations imprinted on the back (flat side) of the frame is a mold error. They are reversed. Please edge lenses accordingly.
- The nylon frame front should be hydrated every 2-3 months by soaking in water for 24 hours. This will help maintain the frame's flexibility.

FRAME FRONT AND BRACKET WARRANTY (excluding lenses): 4 years against breakage if properly cared for. Proper care entails hydrating (soaking) the nylon frame in water for 24 hours every 2-3 months.